




## Manage Device via iOS App

Oximeter Used: CMS50F (Only Device to Upload via iOS/Android platform)

**Login to the App: Click “Manage Devices”**

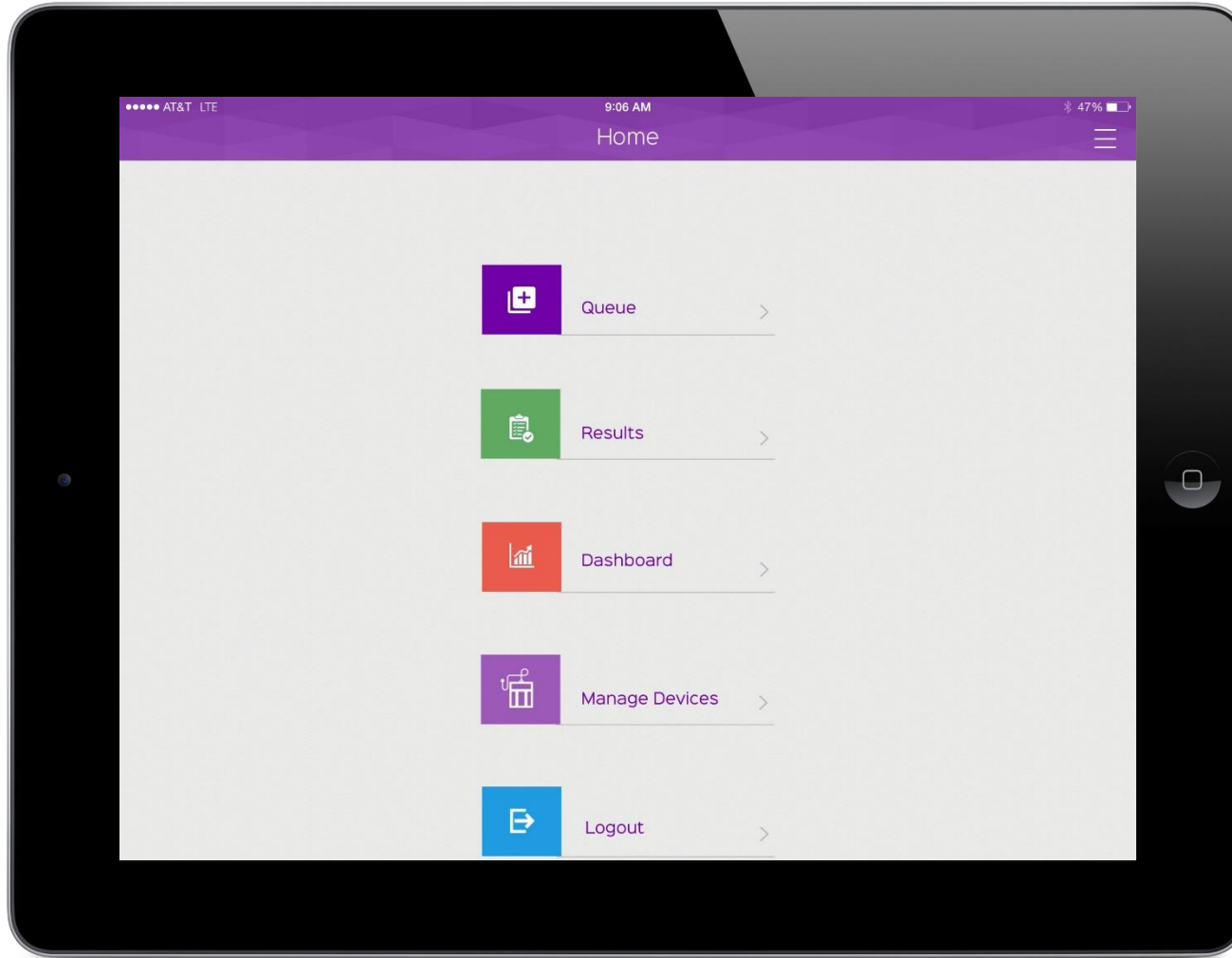




## Manage Device via iOS App

Oximeter Used: CMS50F (Only Device to Upload via iOS/Android platform)

**Set Date/Time: Turn on the oximeter and Click “Set Date & Time” to proceed**

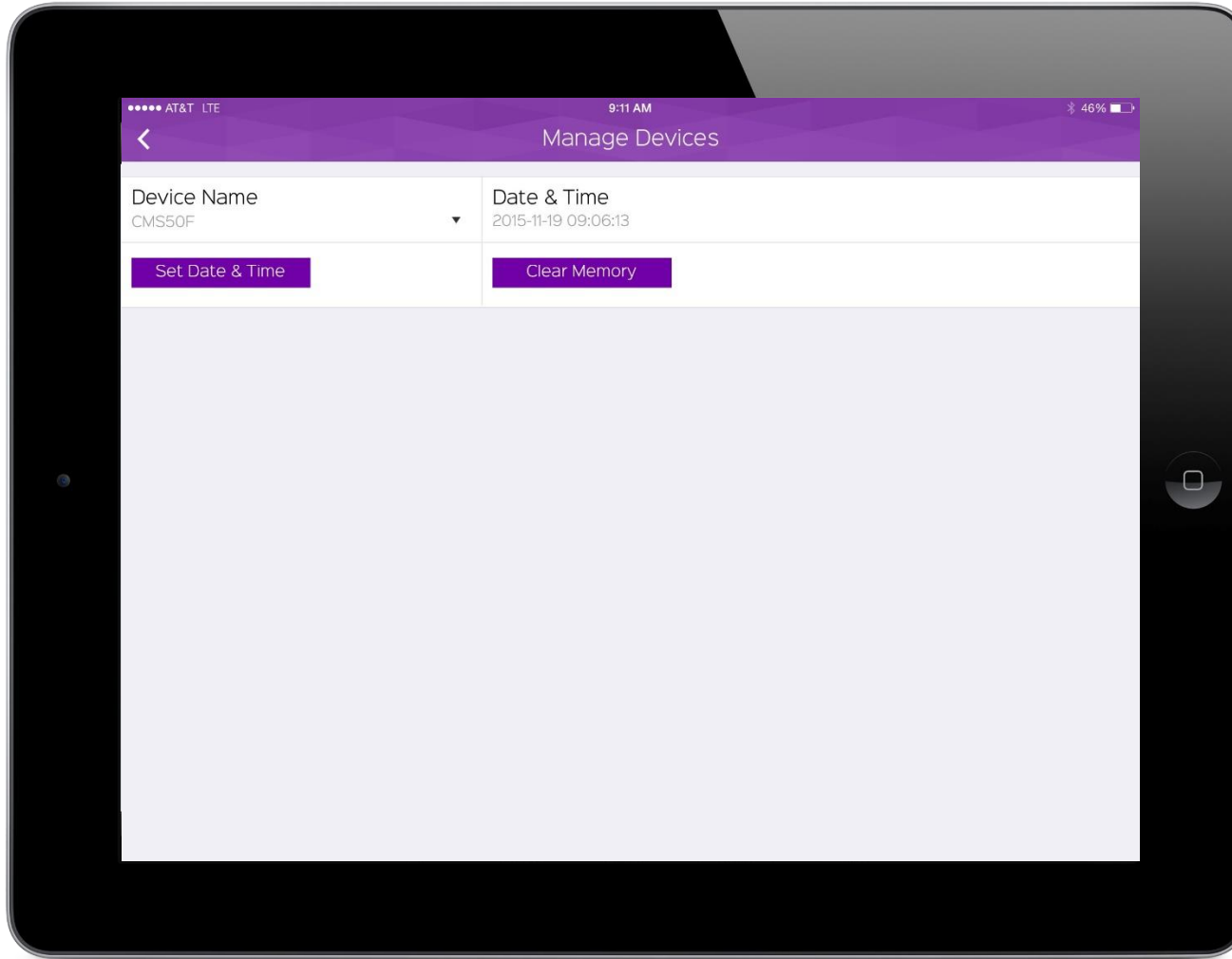




## Manage Device via iOS App

Oximeter Used: CMS50F (Only Device to Upload via iOS/Android platform)

**Click “Ok”:** Date and Time have been saved to the device

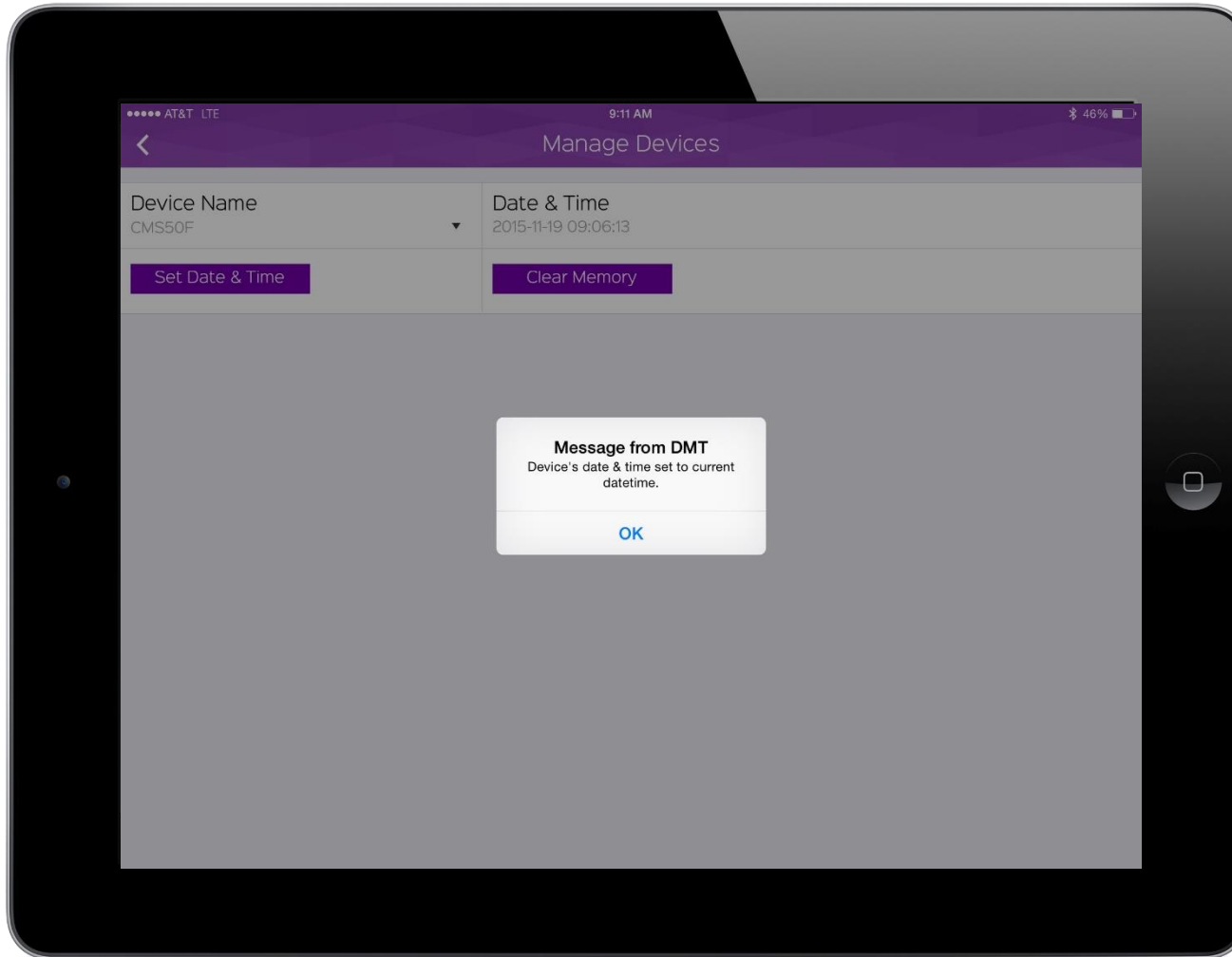




## Manage Device via iOS App

Oximeter Used: CMS50F (Only Device to Upload via iOS/Android platform)

### Clear Memory on Device: Click “Clear Memory”

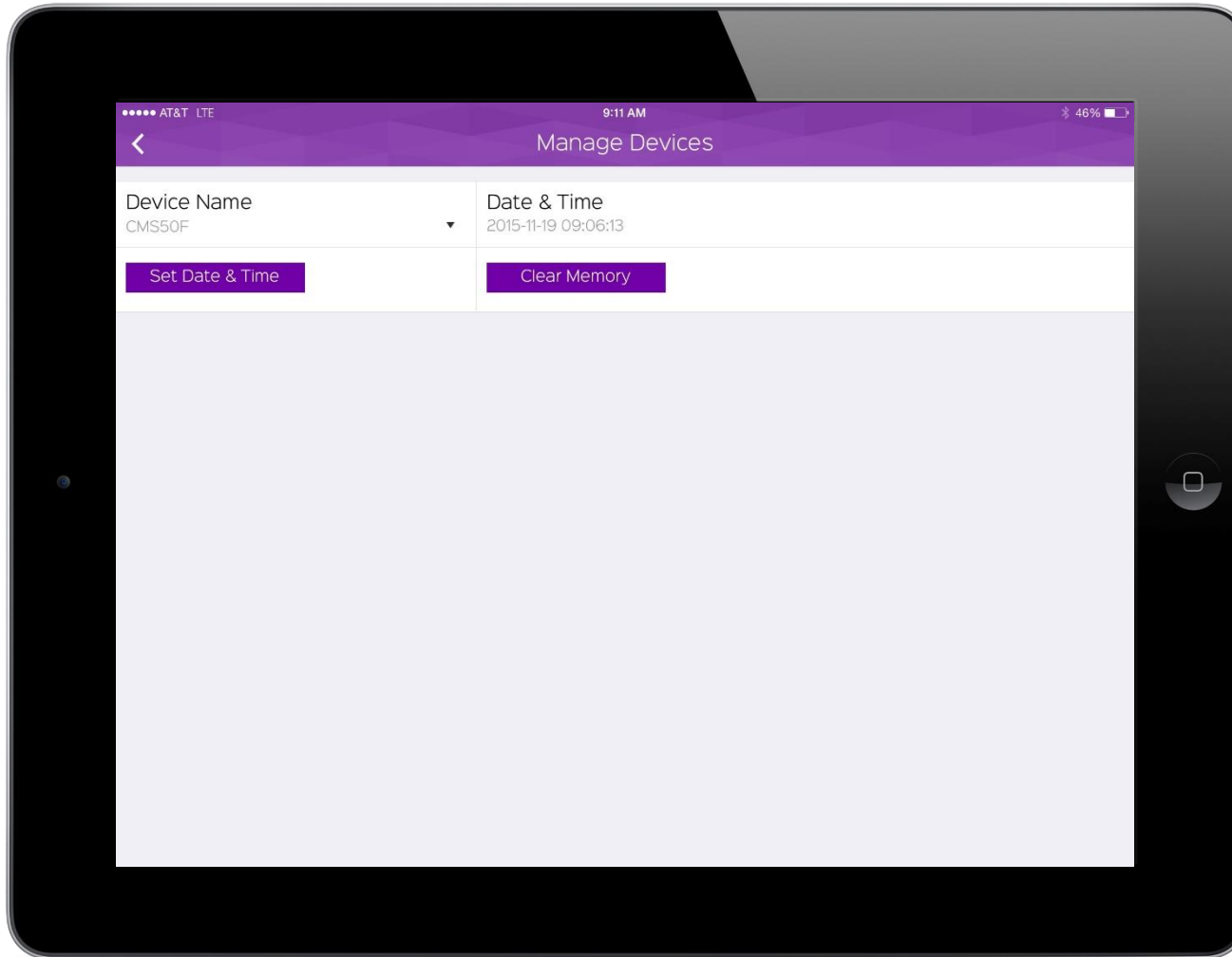



## Manage Device via iOS App

Oximeter Used: CMS50F (Only Device to Upload via iOS/Android platform)

**Click “Ok”:** Device Memory has been cleared from the device

